# 

## EAR IRRIGATION SYSTEM



- Large Volume
- Fast & Simple
- Less Mess Than Syringes

#### **Use to Remove:**

- Wax / Cerumen
- Foreign Bodies



### EARCAP

#### **EAR IRRIGATION SYSTEM**



Use the EarCap for flushing out wax and foreign bodies. The EarCap twists on any standard saline bottle for large volume irrigation. The built in catch basin collects draining fluid before it creates a mess, more work, and a wet uncomfortable patient. The positioning of the head tilted downward, uniquely works with gravity. Draining fluid and debris easily exits down and out of the ear canal. Place the patient next to a sink for self administration, saving your staff time and reducing their wait times. Avoid the tedious refilling of syringes with adapters that sometimes cause deep irritation or pressure. Avoid frightening patients with syringes whicht interfere with cooperation and success. Best of all, your progress can be monitored and rewarded, as your treasures plop into the transparent "see what you get" catch basin.

Always check fluid for appropriate warm temperature. Never use hot or cold fluids. Never irrigate if a tympanic membrane is suspected or known to be ruptured. The off-center nozzle allows a smaller shield to fit over ear. Orient shorter shield side forward. Use a new container or refill the original with clean tap water, as needed. Try the EarCap and "stick it in your ear!"







Purchase Just the Cap. Fits Any Standard Saline Bottle. Adjustable Depth Tubing. Tilted Position for Insertion and Gravity Drainage.

**A Comfortable** Alternative to **Traditional Ear Irrigation** 

**Use to Remove** Wax / Cerumen & **Foreign Bodies** 

**Built-In Catch Basin** 

See What Has **Been Removed** 

Advantages:

- Simple Set Up
- Less Mess
- Disposable
- Inexpensive
- Self-Administration
- Time Saving

In some healthcare settings, ear canal irrigation may be a billable procedure. Consult with your billing professional.

