
## **NDSECAP**<sup>™</sup> NASAL IRRIGATION SYSTEM

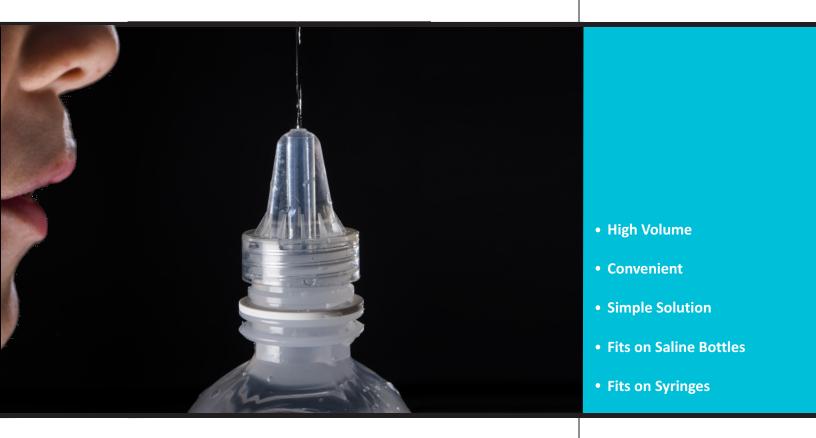



## **NDSECAP**<sup>™</sup> NASAL IRRIGATION SYSTEM

Provide a "solution" to your patients for their cold and allergy symptoms with the NoseCap. Just twist on any standard sterile saline bottle and squeeze to flush out the nasal passages. Using premixed sterile saline in a bottle, avoids the risks that come with using contaminated water sources. It is more convenient than boiling tap water and then mixing with salt packets. For additional control of flow and pressure, an option also exists for using a luer tip syringe. Adults can use 30ml to 60ml luer syringes. For young children, limit use to 5-10ml syringes. The NoseCap system makes it easy to be compliant with nasal irrigation recommendations.



Assemble tube to cap for bottle use. Tilt head forward or to the side. Insert in upper nostril. Squeeze bottle and flush nasal passages. Use in the clinic or hospital; and then demonstrate how to use the NoseCap at home. Consider prescribing saline for home use or patients can continue using their over the counter mixtures at appropriate concentrations, if needed.

