

# **EARCAP™**

## **EAR IRRIGATION SYSTEM**



- Fits on Saline Bottles
- Large Volume
- Fast & Simple
- Less Mess Than Syringes
- Flush Out Wax / Cerumen
- Rinse Out Foreign Bodies

# EARCAP™

## EAR IRRIGATION SYSTEM



- Large Volume Irrigation
- Built In Catch Basin
- Gravity Assisted Drainage
- Adjustable Flexible Eccentric Tubing
- Wide Comfort Brim & Backstop
- Universal Bottle Threads

Use the EarCap for flushing out wax and foreign bodies. The EarCap twists on compatible standard saline bottles for large volume irrigation. The built in catch basin collects draining fluid before it creates a mess, more work, and a wet uncomfortable patient. The positioning of the head tilted *downward*, uniquely works with gravity. Draining fluid and debris easily exits down and out of the ear canal. Place the patient next to a sink for self administration, saving your staff time and reducing their wait times. Avoid the tedious refilling of syringes with adapters that sometimes cause deep irritation or pressure. Avoid frightening patients with syringes which interfere with cooperation and success. Best of all, your progress can be monitored and rewarded, as your treasures plop into the “see what you get” catch basin.

Always check fluid for appropriate warm temperature. Never use hot or cold fluids. Never irrigate if a tympanic membrane is suspected or known to be ruptured. The off-center nozzle allows a smaller shield to fit over ear. Orient shorter shield side forward. Use a new container or refill the original with clean tap water, as needed. Try the EarCap and “stick it in your ear!”



Purchase Just the Cap. Fits Compatible Saline Bottles. Adjustable Depth Tubing. Tilted Position for Insertion and Gravity Drainage.

A Comfortable  
Alternative to  
Traditional  
Ear Irrigation

Use to Remove  
Wax / Cerumen &  
Foreign Bodies

Built-In Catch Basin

See What Has  
Been Removed

Advantages:

- Simple Set Up
- Less Mess
- Disposable
- Inexpensive
- Self-Administration
- Time Saving

In some healthcare  
settings, ear canal  
irrigation may be a  
billable procedure.  
Consult with your  
billing professional.